

**First week:** Use (1) pouch per operatory every day. This is the initial shock treatment.

**Thereafter:** Use (1) pouch per operatory, twice per week. This is the regular ongoing maintenance.



1. Remove dispenser cap and add (1) BioGuard pouch.



2. Fill the plastic dispenser with 720ml of room temperature tap water (measuring indicator on the side of the dispenser label). Then screw cap back on Tip & Pour Dispenser.



3. With suction handpiece valves closed, Attach the high volume suction handpiece (HVE) onto the large plastic tubing and the saliva ejector (SE) onto the smaller tubing. Once secure, simultaneously open each valve to allow container mixture to be sucked up through the lines. Container should empty in approximately 30 seconds.



4. Repeat steps 1-3 for each operatory

*\*Make sure to check the filter on your vacuum pump after switching to BioGuard. BioGuard will remove excess buildup from the lines that had not been removed with other cleaners.*