GB

## Almore Bite Registration Wax Tabs

---

### Instructions for use

---

Almore Bite Registration Wax was developed to meet your needs of accurate occlusal records. Our wax is so accurate that it records the details of the occlusion exactly. When pouring up models, avoid discrepancies between the two materials by trimming the bite record so only the cusp tip indentations are present.

Heat in a thermostatically controlled waterbath at 135° Fahrenheit (57.2° Celsius) for one minute. After obtaining jaw relation records, place the record in cold water to regain maximum hardness and to facilitate trimming of excess wax.

If the Almore Bite Registration Wax is not performing as expected, switch to alternative method.

**Warning!** Hot wax can cause burns to eyes and skin. When handling hot wax, use heat-protective gloves and other PPE to protect against thermal burns. Spills may create a slipping hazard.

The manufacturer and the competent authority of your member state must be immediately notified of all serious incidents occurring in connection with the product.

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