

Cleaning and Sterilization Instructions

HVE / Saliva Ejector

Hager Worldwide Products are provided non-sterile

Ultrasonic Cleaner

- Place instrument in the Ultrasonic Cleaner until all bioburden is removed
- Use neutral PH solution for cleaning instrument
- Thoroughly rinse all solution from instrument
- Allow to dry

Instrument Washer

- Only Use Instrument Washer certified for use by FDA for Medical Dental Use
- Use neutral PH cleaner designed for use in Instrument Washer
- Run Washer for complete cycle

Physically Inspect O-rings

- Replace o-rings if they appear to be cracked or damaged

Sterilization

- Steam Sterilization is the recommended method
- Always consult and follow the sterilizer manufacturers instructions and recommendations

Cycle Type	Minimum Temperature	Minimum Exposure Time	Drying Time
Pre-vacuum	121 C 250 F	4 minutes	30 minutes
Gravity Feed	121 C 250 F	30 minutes	30 minutes