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Choose your preferred placement option!

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Heel of the hand
Talon de la main
Talón de la mano

Palm of the hand
Paume de la main
Palma de la mano

Distributed By:
Hager Worldwide, Inc.
441 19th Street SE
Hickory, NC 28602
(800) 328-2335

Fit-N-Swipe CLEAN

D24860525101

REF 605 251

Fit-N-Swipe DRY

D24860525202

REF 605 252

Patent Pending
MED-RAS GmbH
Eichenallee 8H
21521 Wohltorf, Germany

See the Fit-N-Swipe in Action!

Fit-N-Swipe

Adhesive cleaning pad for cleaning dental instruments during a procedure

Fit-N-Swipe
CLEAN
(White Pad)

Fit-N-Swipe
DRY
(Blue Pad)

50

single-use pads
tampons à usage unique
almohadillas de un solo uso

(800) 328-2335 | www.hagerworldwide.com

Fit-N-Swipe Clean

Adhesive cleaning pad for clean instruments

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Fit-N-Swipe Dry

Adhesive drying pad for crystal clear mouth mirrors

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Instructions for use

Sterilization:

Pads are **NOT STERILE** and can be sterilised in a certified autoclave according to instructions for textiles materials **BEFORE APPLIANCE**

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Method	Process	Temperature	Min. Holding Times	Drying Times
Hot Steam	Vacuum Process	132°	4 minutes	20 minutes
	Fractionated Pre-vacuum	135°	3 minutes	

Removal of the silicon paper with gloves:

- First bend the Pads for better removal
- DO NOT touch adhesive surface – unintentional extensive contact of the adhesive surface significantly reduces bond

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Positioning of the Pads:

- Pad must be firmly pressed - pad sticks only once – if changing the position pad must be replaced by a new one
- **DO NOT** glue Pad directly onto the skin, glue is not tested dermatologically
- All surfaces (gloves/rubber dam) where pads should be stuck on **HAVE TO BE ABSOLUTELY DRY**
- Any disinfectant or soap on rubber dam or gloves must be removed before application of the Pad - depending on the kind of disinfectant or soap used, the sticking strength can be reduced due to change of the surface property of the rubber dam or the glove material

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Adhesion on the gloves:

- The adhesion is approved on Vinyl, Nitrile and Latex gloves powdered and un-powdered – if for any reason the bonding should be reduced due to specific production processes of the gloves we kindly ask you to contact the provider of the pads for further assistance

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