
# Mirror Suction “Double Sided” Instructions for Use

## Usage

- Remove protective film from both sides of the mirror head.
- Insert plain end of tubing onto adapter
  - It is helpful to place tubing into warm water before attaching to adaptor
- Place adapter into the High Volume Evacuator (HVE) handpiece
- Screw the mirror handle into the mirror handle
- Place Mirror Handle into connector on tubing
  - Lubricate “O” Ring in connector weekly
- Turn HVE handle to open to start suction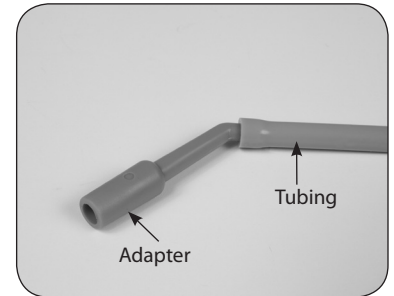

## Maintenance and Sterilization

- Adapter, tubing, handle and mirror are sterilizable

## Ultrasonic Cleaner

- Place in Ultrasonic Cleaner until all bioburden is removed
- Rinse cleaning solution from instrument thoroughly with soft water
- It is most effective to disassemble mirror and handle from tubing before ultrasonic cleaning.
- Thoroughly dry instrument before bagging

## Instrument Washer

- Run instrument washer based upon manufacturer’s instruction
- Only use Instrument Washer certified by FDA for medical/dental use
- Use neutral pH cleaner certified for use in Medical Dental Instrument Washer


## Sterilization

Cycle Type	Minimum Temperature	Minimum Exposure Time	Drying Time
Pre-vacuum	121° C 250° F	4 minutes	30 minutes
Gravity Feed	121° C 250° F	30 minutes	30 minutes

*\*Recommended to sterilize in Cassette*

## Maintenance

- Lubricate “O” Ring in connector with silicone lubricant
- Separate mirror from handle and clean threads occasionally
- Clean Mirror Surface with soap and water

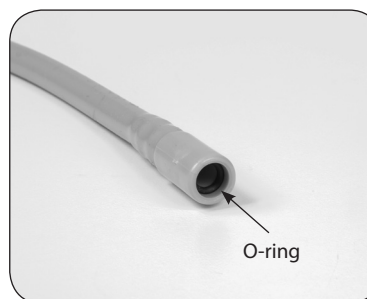*For more detailed cleaning and care information please call Hager Worldwide customer service at 800-328-2335.*

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