
Mirror Suction “Single-Sided” Instructions for Use

Usage

- Remove protective film from the mirror head.
- Insert plain end of tubing onto adapter
 - It is helpful to place tubing into warm water before attaching to adapter
- Place adapter into the High Volume Evacuator (HVE) handpiece
- Screw the mirror into the mirror handle
- Place mirror handle into connector on tubing
 - Lubricate “O” Ring in connector weekly
- Turn HVE handle to open to start suction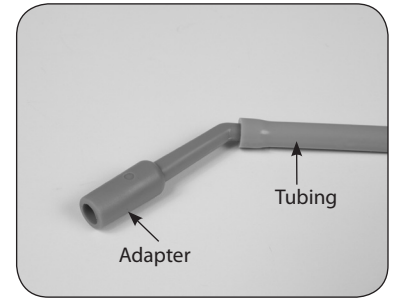

Maintenance and Sterilization

- Adapter, tubing, handle and mirror are sterilizable

Ultrasonic Cleaner

- Place in Ultrasonic Cleaner until all bioburden is removed
- Rinse cleaning solution from instrument thoroughly with soft water
- It is most effective to disassemble mirror and handle from tubing before ultrasonic cleaning.
- Thoroughly dry instrument before bagging

Instrument Washer

- Run instrument washer based upon manufacturer’s instruction
- Only use Instrument Washer certified by FDA for medical/dental use
- Use neutral pH cleaner certified for use in Medical Dental Instrument Washer


Sterilization

Cycle Type	Minimum Temperature	Minimum Exposure Time	Drying Time
Pre-vacuum	121° C 250° F	4 minutes	30 minutes
Gravity Feed	121° C 250° F	30 minutes	30 minutes

**Recommended to sterilize in Cassette*

Maintenance

- Lubricate “O” Ring in connector with silicone lubricant
- Separate mirror from handle and clean threads occasionally
- Clean Mirror Surface with soap and water

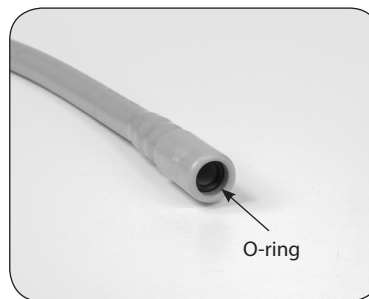For more detailed cleaning and care information please call Hager Worldwide customer service at 800-328-2335.

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