

# Cleaning and Sterilization Instructions

## PlastFrame

Hager Worldwide Products are provided non-sterile

### Ultrasonic Cleaner

- Place instrument in the Ultrasonic Cleaner until all bioburden is removed
- Use neutral PH solution for cleaning instrument
- Thoroughly rinse all solution from instrument
- Allow to dry

### Instrument Washer

- Only Use Instrument Washer certified for use by FDA for Medical Dental Use
- Use neutral PH cleaner designed for use in Instrument Washer
- Run Washer for complete cycle

### Sterilization

- Steam Sterilization is the recommended method
- Always consult and follow the sterilizer manufacturers instructions and recommendations

Cycle Type	Minimum Temperature	Minimum Exposure Time	Drying Time
Pre-vacuum	121 C 250 F	4 minutes	30 minutes
Gravity Feed	121 C 250 F	30 minutes	30 minutes