



## PLEASE READ THESE IMPORTANT INSTRUCTIONS BEFORE USING YOUR PRESSURE POT.

### INSPECTION



After unpacking your pressure pot please inspect thoroughly for any damage that may have incurred during shipping.



Inspect around the lid gasket to make sure there is no debris that will inhibit proper sealing.

If you find any damage please do not use the pressure pot and please return to your dealer.

### LID SEATING



When installing the lid place lid onto pot lining up the two triangles.



Press firmly and evenly on the lid to make sure it is properly seated.



Assure that the lid moves freely without binding. If your lid does not move freely, please do not use the pot and return to your dealer.

### ATTACHING THE AIR LINE



When connecting airline to the pressure valve, hold the valve **firmly** and connect the airline. This will help avoid any damage to the valve assembly.

Use only a standard air compressor. **DO NOT** attempt to use a manual air pump.

VIDEO  
INSTRUCTIONS

