



HAGER
WORLDWIDE
Dental Products Worldwide

441 19th Street SE
Hickory, NC 28602
Tel: (800) 328-2335
Fax: (800) 573-9392
www.hagerworldwide.com

Cleaning and Sterilization Instructions

Super Bib Clips

Hager Worldwide Products are provided non-sterile

- Place Super Bib Clips in the Ultrasonic Cleaner or Instrument Washer.
- Run complete cycle based upon Ultrasonic Manufacturer Instruction.
- Rinse items thoroughly to remove all cleaning solution.
- Bag Super Bib Clips and sterilize based upon the sterilizer manufacturer's instructions.
- Recommended time is 4 minutes or complete cycle at 270 degrees.
- http://www.ada.org/sections/professionalResources/pdfs/cdc_sterilization.pdf