

Cleaning and Sterilization Instructions

1. Place Svedopter (254140) in Ultrasonic Cleaner or Instrument Washer.
2. Run complete cycle based upon Ultrasonic Manufacturer Instruction.
3. Rinse device thoroughly to remove cleaning solution completely.
4. Bag instruments and sterilize based upon the sterilizer manufacturer's instructions.
5. Recommended time is 4 minutes cycle at 270 degrees
6. http://www.ada.org/sections/professionalResources/pdfs/cdc_sterilization.pdf