CROWN CLICK DE LUXE

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CROWN CLICK DE LUXE

Instructions for use

Crown Click de Luxe is a mechanical crown and bridge remover for the removal of fixed and temporary dentures.

- By an intensive impulse of the impact stress inside the instrument, the cement layer breaks and the denture can be removed easily. The periodontium does not get damaged in this process.
- Easy handling: One hand holds the instrument, the other hand fixates the hook top at the denture with practically no assistance needed.
- The infinitely adjustable impact intensity as well as three different hooks guarantee a universal use.

Function

The impact strength is adjusted by turning the knurled-head screw (visible in the empty space of the instrument shelter).

- Stronger: Turning of the knurled-head screw towards the hook – raising of spring tension
- Weaker: Turning of the knurled-head screw towards the trigger – releasing of spring tension

Change of instruments

The hooks can be exchanged easily and fast. Put the delivered fork wrench on the groove of the hook fixing aid and penetrate into the drill of the receiver hook with the guide pin. By holding the fixing tube with the fork wrench and by a left turn of the guide pin, the concerned hook can be taken out easily. The complete device is made of high-quality stainless steel and practically maintenance-free.

Scope of supply

Set: Device with all three instruments (incl. fork wrench, guide pin for mounting and demounting the hooks)

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Medical device: Crown Click De Luxe

REF 452 205 - 452 209

Warning notices: No special notices

Limitation of reprocessing:

Frequent reprocessing has only a small impact on this instrument. The instrument's shelf life is mainly determined by wear and damage by use.

Place of handling:

Clean surface with a disposable cloth or paper towel.

Storage and transport:

No special requirements.

Preparations for cleaning:

Take the product to pieces.

Cleaning:

Use cleaning agent suitable for stainless steel carefully following the instructions of the respective manufacturer.

After cleaning, remove cleanser thoroughly under running water. Do not exceed 93 °C for mechanical cleaning.

Disinfection:

Use disinfecting solution suitable for stainless steel carefully following the instructions of the respective manufacturer.

Remove disinfecting solution thoroughly under running water.

Maintenance:

No special requirements.

Packaging:

Standardized packing material may and should be used for sterilisation process.

Sterilisation:

Vapor sterilisation at 134 °C for 5 minutes or vapor sterilisation at 121 °C for 15 minutes.

Control/Functional check:

Sight check on damages, wear, deformation.

Storage:

No special requirements.

The a.m. instructions have been validated as SUITABLE for preparation of a medical device and its re-use by the medical device manufacturer. It is the reprocessor's responsibility that the actual reprocessing achieves the required results with the equipment and materials applied and personnel involved in the reprocessing site. Normally, validation and routine control are necessary. Furthermore, the reprocessor should evaluate any deviation from the provided instructions regarding efficiency and possible adverse consequences.